STERILIZATION TYPE 2 BOWIE-DICK

EXPOSED

UNEXPOSED

WHAT IS BOWIE-DICK TEST PACK STEAM CHEMICAL INDICATOR?

This indicator is used to ensure the correct operation of the autoclave. This test is performed before the start of the working day and sterilization with a steam autoclave. The method of using this test is that first you have to place the test alone in the middle of the bottom shelf of the steam autoclave device and then adjust the device according to the Bowie-Dick test being used. The settings related to the Bowie Dickpackage test are temperature 134 degrees and time 3.5 minutes. If the test is successful, the device can be used to sterilize tools, otherwise the device is in trouble. This test can be archived and it can be archived along with the date of the test.

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QUICK TIPS! Unused indicators should be stored in their original packaging.

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QUICK TIPS! Store at room temperature (10-38°C) and 10-70% humidity, away from oxidizing agents, disinfectants, and other liquids.

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QUICK TIPS! Keep out of direct sunlight. No special storage required post-sterilization.

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Tel: +982141269999
info@krinmed.com
www.krinmed.com

Khosrow 50
South Villa
Tehran - Iran

Made in Iran
by Krinmed Inc
member of AltonTeb group