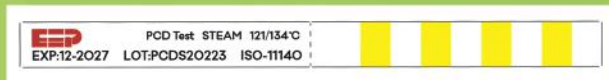
**EXCELLENT ELITE PROCESS
KRINMED INC.
COMPANY**

STERILIZATION TYPE 2 CHEMICAL INDICATOR

EXPOSED

UNEXPOSED

WHAT IS

PCD STEAM CHEMICAL INDICATOR?

The PCD test is used as a daily test for pre-vacuum hospital steam sterilizers. This is a class 2 indicator and is used to sterilize lumen instruments and tubes using a steam autoclave. When performing the cycle with this indicator, you must fold it from the place marked on it and place it inside the PCD indicator holder. Then put the relevant instruments together with the steam PCD chemical indicator in the steam sterilizer. In this test, if all the houses on the indicator change to the final color, the cycle has been completed successfully, and if the color change of even one of the houses is different from the final color, the cycle has encountered an error and the test must be repeated.

Unused indicators should be stored in their original packaging.

Store at room temperature (10-38°C) and 10-70% humidity, away from oxidizing agents, disinfectants, and other liquids.

Keep out of direct sunlight. No special storage required post-sterilization.

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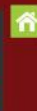
www.krinmed.com

Khosrow 50
South Villa
Tehran - Iran

Made in Iran
by Krinmed Inc
member of AltonTab group

STEAM